



DermaLoops GFC Tubes

Sterile, Single-Use GFC Tubes for Hair & Skin Treatments

Our GFC Tube Pouch Pack contains premium-quality tubes designed for dermatologists and trichologists. Individually sealed for safety and hygiene, this pouch allows clinics to easily perform hair restoration and skin rejuvenation treatments with confidence and convenience.



www.dermaloops.com

9400323106

dermaloops@gmail.com

Why Choose Dermaloops GFC Tubes Pouch Pack?

Sterile and single-use, ensuring safety and ease of handling

Compact, travel-friendly pouches for clinics of any size

Consistent, high-quality tubes to support effective hair and skin therapies

Designed to simplify clinical procedures with minimal preparation

Benefits & Usage

Ready-to-use sealed tubes for smooth, contamination-free treatments


Delivers a high yield of growth factors for hair regrowth and skin rejuvenation

Trusted by dermatologists and aesthetic professionals for their reliability

INSTRUCTION FOR GFC PREPARATION

12ML

STEP 1

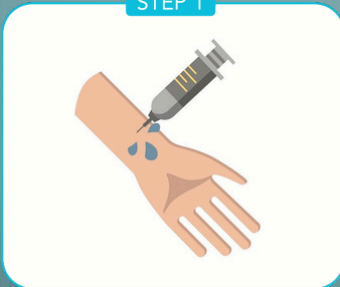

COLLECTION OF BLOOD

Collect 10ml of blood in each tube


STEP 2



ACTIVATION OF GFC

Mix by Inverting gently 8-10 Times and allow to Stand upright for 30 Minutes

STEP 3

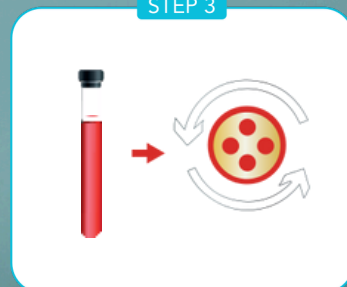

SEPARATION OF GFC

Centrifuge all the Tube at 3500 RPM for 10 Minutes


STEP 4



COLLECTION OF GFC

Collection of GFC Serum from the tubes into the Syringes (up to 7.5ml per tube)

STEP 5

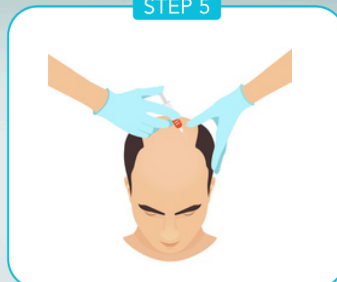

APPLICATION

GFC is ready to Inject.